

Dynasyst

Coils & springs

GENERAL PRODUCT INFORMATION

Dynasyst springs are made of steel wire according to the DIN EN 10270-1 standard.

The coil designs are made by professionals according to information gained from therapeutic protocols.

Orfit coils and springs are designed to offer support for a variety of upper extremity joints injured due to nerve damage or soft tissue damage. We offer coils for the wrist, MCP joints, PIP joints, and also spring wire which can be fashioned into custom-made coils according to the patient's needs.

PRODUCT RANGE

Dynasyst springs are available in different pre-coiled spring wires:

Article Number	Description 1 BC
35800	Finger Coil Springs - 10 pcs
35801	Knuckle Bender Coil Springs - 10 pcs
35802	Finger Extension Assists - 10 pcs
35804	Wrist Extension Coil Springs - 10 pcs
35805	Stainless Steel Spring Wire

PRECAUTIONS BEFORE USE

WORKING TECHNIQUE

1. Make sure at least 1 leg of the coil is bent in a U-shape intended to be sandwiched between 2 layers. Avoid sharp angles and notches that would cause break points.
2. Place the coil so that active patient motion winds up the coil.
3. Position the coil if possible aligned with the physiological axis.
4. To fix the coil arm: heat it up first, then melt it into the plastic layers.
5. Bend the free coil leg also in a U-shape.
6. Adjust the coil strength by augmenting or diminishing the number of coils with spring coil pliers.
7. To replace a coil, heat up the coil arm, detach from the sandwich fixation and replace.

GENERAL SAFETY ADVICE

1. Please exercise caution when cutting the springs.
2. Heating the coils can cause burns to your hand.

ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets, and regulatory information, please visit our website www.orfit.com.

Note:

It is prohibited to make alterations to this text without prior approval from Orfit Industries.
This product is a registered trademark from ORFIT INDUSTRIES BV.

**ORFIT INDUSTRIES BV**

Vosveld 9A | B-2110 Wijnegem | Belgium

T (+32) (0)3 326 20 26

welcome@orfit.com

www.orfit.com



Ref. Nr.	31210
Version	57
Last updated	1/12/2024
Revision date	1/12/2026