

## Padding and Lining Orfilastic

### GENERAL PRODUCT INFORMATION

Orfilastic is an elastic terrycloth lining material with a backing of self-adhesive open cell foam. It is used to improve air transmission of orthoses, external immobilization devices and rehabilitation aids and to ameliorate the comfort of wearing these products.

### PRODUCT RANGE

Orfilastic is available in different colors with a size of 50 cm (width) x 100 cm (length) x 3 mm (thickness).

Article Number	Description 1 BC
35111	Orfilastic Self-Adhesive
35117	Orfilastic Self Adhesive Black 50 x 100cm x 3mm

### PRECAUTIONS BEFORE USE

- Cut the pattern before removing the release paper from the back (direction has no importance).
- Orfilastic is fixed on the inside of the orthosis, after the orthosis has been formed.
- Remove the release paper and stick Orfilastic on the required area.
- Proceed with removing the paper while sticking Orfilastic in the orthosis under slight pressure. The pressure must avoid the emerging of folds.
- Make sure that Orfilastic does not stick back to back: this is irrevocable.

### MAINTENANCE

- Orfilastic is NOT waterproof and has to be replaced when necessary.

### WASTE MANAGEMENT

- After use, the product can be disposed of with normal household waste without harming the environment.

### ADDITIONAL INFORMATION

For additional information such as distributor contact information, product brochures, Safety Data Sheets, and regulatory information, please visit our website [www.orfit.com](http://www.orfit.com).

**Note:**

It is prohibited to make alterations to this text without prior approval from Orfit Industries.  
This product is a registered trademark from ORFIT INDUSTRIES BV.



**ORFIT INDUSTRIES BV**

Vosveld 9A | B-2110 Wijnegem | Belgium

T (+32) (0)3 326 20 26

welcome@orfit.com

[www.orfit.com](http://www.orfit.com)



Ref. Nr.	31301
Version	14
Last updated	1/12/2024
Revision date	1/12/2026