# VACUUM BAG CUSHIONS

| Article Nos.:           |           |      |
|-------------------------|-----------|------|
| GENERAL PRODUCT INFORMA | ATION     |      |
|                         |           |      |
|                         |           |      |
|                         |           |      |
| PRODUCT DESCRIPTION     |           |      |
|                         |           |      |
|                         |           |      |
| HOW TO USE              |           |      |
|                         |           |      |
|                         |           |      |
|                         |           |      |
|                         |           |      |
|                         |           |      |
|                         | Waterbath | Oven |
|                         |           |      |

#### **STORAGE**

Always store the products in a safe place to prevent them from getting damaged. Store the products in such a way that they cannot fall or bump against anything. Especially, contact with sharp edges or objects must be avoided as they may damage the fabric of the vacuum bags.

The vacuum bags can be stored using the hole to hang them on a hook. When not used, the vacuum bags should be evaluated and stored on a flat surface with their bottom side (orange fabric) down. Do not stack the evacuated vacuum bags on top of each other.

When not in use, store the vacuum deflated or under a mild vacuum. This will increase the lifespan of the vacuum bags.

Store the devices between +10°C (50°F) and 40°C (122°F) in a place that is not humid.

Always store the products in a safe place to prevent them from getting damaged. Store the products in such a way that they cannot fall or bump against anything. Especially, contact with sharp edges or objects must be avoided as they may damage the fabric of the vacuum bags.

The vacuum bags can be stored using the hole to hang them on a hook. When not used, the vacuum bags should be evaucated and stored on a flat surface with their bottom side (orange fabric) down. Do not stack the evacuated vacuum bags on top of each other.

When not in use, store the vacuum deflated or under a mild vacuum. This will increase the lifespan of the vacuum bags.

Store the devices between +10°C (50°F) and 40°C (122°F) in a place that is not humid.

# Maintenance Management

These products can be cleaned and disinfected by means of an isopropanol based disinfectant, applied with a soft cloth. If unsure about the cleaning fluid, do not use. Never use aerosol sprays, corrosive cleaning agents, solvents or abrasive detergents. Do not soak the products.

Further cleaning instructions can be found in the Orfit Cleaning Guidelines.

Periodic checks of these products should be done to insure the parts are not worn and require repair or replacement. Repair kits for both the orange and grey fabric are available. Contact your distributor if there are any questions or concerns.

## **Waste Management**

The product can be disposed of with household waste.

## **ADDITIONAL INFORMATION**

For additional information such as distributor contact information, product brochures, Safety Data Sheets and regulatory information, please visit our website <a href="https://www.orfit.com">www.orfit.com</a>.

Note: